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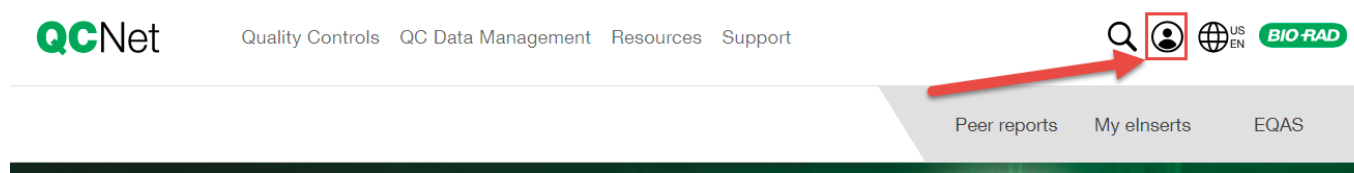
Register

Some benefits for registering

- Receive Unity peer reports
- Register lab profile
- Set up EQAS online account
- Access barcode generator, linearity, my elnserts XML

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Email Verification

Dear Test_1

Thank you for joining QCNet. To finish registering, please check your inbox for an email from **unity_support@bio-rad.com** and click on the verification link.

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Email Verification

Dear Test_1,

Click on the button below to verify your email

Verify Email

If the button did not work, please copy this url into your web browser:

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BIO-RAD

Thank You

Your email has been verified. You may log in now.

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EQAS Online

The screenshot shows the 'EQAS Online Account Request' form. At the top, a welcome message states: 'Welcome to the EQAS Online Proficiency testing reporting software. You do not have an EQAS Online account set up at this time, please enter your Lab number in the bottom field so Bio-Rad can set up your account, please allow a couple of days for your account to be created, Bio-Rad will send you an email when your account is available.' The form contains several input fields: 'First Name', 'Last Name', 'Email', 'Phone Number', 'Extension', 'Fax Number', 'Company', 'Street', 'City', 'State', 'Postal Code *', and 'Country' (a dropdown menu currently showing 'Albania'). Below these is a section for 'Lab' numbers with a note: 'Please enter the lab number you wish to register. If multiple labs, separate with commas.' This is followed by an 'Additional comment' field. At the bottom are 'Cancel' and 'Submit' buttons.

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